



Simian Quantum  
Innovations, Inc. 

7961 Otway Rd. Prince George,  
BC, V2M 7B4, Canada  
1 (250) 564-9011  
1 (250) 961-2226  
info@simianquantum.ca  
www.simianquantum.ca  
www.magnemasque.ca

## MagneMasque™ Products

The MagneMasque™ by Simian Quantum is the world's only face mask that utilizes our patent pending magnet-infused MagneStraps™ which allows for QuickClick connection around the back of the head and not the sides - relieving your precious lobes from the unnecessary strain caused by all the other designs, and freeing you up to play the rest of your day by ear.

The MagneMasque™ is a sanitary face mask that has detachable head straps so that the mask may be removed from the straps via connecting snap fasteners and washed separately from the straps; as well as, adjustable straps that are easily anchored around the head, and not via the ears, utilizing two (2) high-strength magnets, which are embedded into the strap material. The magnetic attraction connects the straps around the back of the head therefore anchoring the mask snug to the face. A significant yet minimal amount of force (approx. 2.5lbs), is then required to simply 'pull' or 'separate' the straps apart to remove the mask from the face when required.

*The MagneMasque™ is made up of a face mask with a contour that covers the nose, mouth and chin of the person's face, and contains the following features:*

1. A detachable face mask that protects the wearer from unwanted foreign particles or contaminants; the mask is composed of a canvas that ranges in facial contours - larger for adults and smaller for children/youth - that contains four (4) snap fasteners located at the four (4) corners of the mask used for easily attaching/detaching the mask from the anchor straps.
2. Two (2) adjustable head straps that contain a snap fastener at either end of the strap for ease in removing the mask to wash and reuse, as well as a high-strength magnet that is embedded into the strap material for the QuickClick connection, and is positioned (for comfort), via anti-slip cord buckles which are then connected via magnetic attraction around the back of the head.

Each and every MagneMasque™ is handmade in Canada with the highest quality standards and materials, and undergoes a rigorous quality control process (including UV blacklight inspection), before leaving our factory. The MagneMasque™ is the most convenient, high-quality, stylish and affordable face mask on the market today. The MagneMasque™ comes in several sizes ranging from child size to extra-large, with increasing levels of protection varying from basic 2-ply to high-end triple-ply with activated carbon filters that are ideal for industrial sites, medical fields and sensitive environments.



SCAN ME

*Shoot us a message with any questions at*  
[info@simianquantum.ca](mailto:info@simianquantum.ca)  
Visit [www.simianquantum.ca](http://www.simianquantum.ca) or [www.magnemasque.ca](http://www.magnemasque.ca)  
For More Information.

*Breathe easy, and try your MagneMasque™ today.*



SCAN ME

## Canada COVID-19: How to safely use a non-medical mask or face covering:

### Do's

- Do wear a non-medical mask or face covering to protect yourself and others.
- Do ensure the mask is made of at least 2 or 3 layers, including 2 layers of tightly woven fabric, ideally with a filter or filter fabric between layers.
- Do inspect the mask for tears or holes.
- Do ensure the mask or face covering is clean and dry.
- Do wash your hands or use alcohol-based hand sanitizer before and after touching the mask or face covering.
- Do use the straps to put on and remove the mask.
- Do ensure your nose and mouth are fully covered.
- Do replace and launder your mask whenever it becomes damp or dirty.
- Do wash your mask with hot, soapy water and let it dry completely before wearing it again.
- Do store re-usable masks in a clean paper bag until you wear it again.
- Do discard masks that cannot be washed in a plastic lined garbage bin after use.

### Don'ts

- Don't wear masks with exhalation valves or vents.
- Don't wear a loose mask.
- Don't touch the mask while wearing it.
- Don't remove the mask to talk to someone.
- Don't share your mask.
- Don't leave your used mask within the reach of others.
- Don't reuse masks that are damp, dirty or damaged.

Remember, wearing a non-medical mask or face covering alone will not prevent the spread of COVID-19. Stay at home if you're sick, wash your hands often and practice physical distancing.

### Do your part to protect yourself and others

#### Wear a non-medical mask or face covering:

- When you're in shared indoor spaces.
- When you can't maintain a 2-metre physical distance from others.
- As advised by your local public health authority.

### Be kind

*Some people may not be able to wear a mask or face covering.*

#### Non-medical masks are not recommended for:

- People who suffer from an illness or disabilities that make it difficult to put on or take off a mask.
- Those who have difficulty breathing.
- Children under the age of 2 years.

*Be Environmentally Responsible - Wear Reusable Masks Whenever Possible - Washable and Reusable Masks are More Environmentally Friendly Than Disposable Masks - If You Must Use a Disposable Mask, Dispose of it Properly.*

### Don't Litter.